

SURGI-GEL PLUS™ (LUBRICATING GEL) 45g

Composition	A viscous, water based, sterile adhering gel. Surgi-gel PLUS™ contains Purified Water, Glycerol, Carbomer 940, Propyl Hydroxybenzoate and Methyl Hydroxybenzoate as preservatives. Sterilised by gamma irradiation.
Use	Surgi-gel PLUS™ is a sterile, greaseless, transparent, water-based, non-staining, non-irritating lubricating gel. Recommended for specific surgical and gynaecological lubrication when increased adherence is needed
Contraindications	Known hypersensitivity to Methyl Hydroxybenzoate and Propyl Hydroxybenzoate.
Directions for use	Apply to instruments or directly to mucosal surfaces and orifices as required. Single use only.
Adverse Reactions	Surgi-gel PLUS™ is normally well tolerated, but should skin irritation occur, discontinue use. Sensitisation has occurred when medications containing Hydroxybenzoates have been applied to damaged or broken skin. After topical use, local irritation may occur. Hypersensitivity reactions have been reported.
First Aid	For advice, contact a Poisons Information Centre (Phone Australia 131126; New Zealand 0800 764 766) or doctor.
Storage	Store in a cool dry place out of direct sunlight.
Poison Schedule	Not scheduled using the criteria in the Standard Uniform Schedule for Medicines and Poisons.

Presentation

Code	Product	Size
SUR03029F	SURGI-GEL PLUS™	45g

Contact Point	Perrigo Australia in Australia & Orion Laboratories (NZ) Ltd in New Zealand Orion Laboratories Pty Ltd, trading as Perrigo Australia ABN 56 009 293 136 25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA Telephone (all hours): +618 9441 7800 FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110 EMAIL: customerservice@perrigo.com.au; WEBSITE: www.perrigo.com.au ORION® is a registered trademark of Orion Laboratories Pty Ltd
----------------------	---

END OF USER GUIDE