

# INHALO<sup>®</sup>

## Medical Oxygen

**Step by step** Operating instructions



# Step by step operating instructions – INHALO®

## Before use:

Remove the round grey tamper evident seal

1




2

Open the grey hinged cover to expose tubing connection



4

Open cylinder valve slowly by turning hand wheel anti clockwise (until it stops) and 1/4 turn back

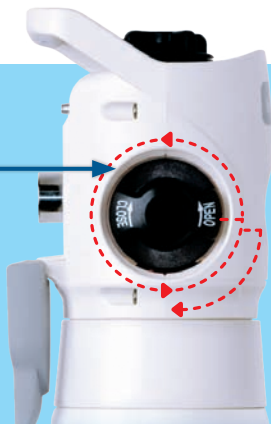

5

Turn flow selector knob to flow rate required



INHALO® now ready for use

# Step by step operating instructions – INHALO®

3


Connect  
Oxygen tube  
to connection



## After use:

1

When Oxygen  
is no longer  
needed turn off  
cylinder valve

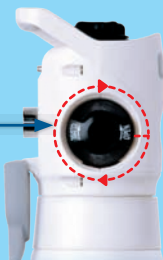

2

Turn flow  
selector knob  
to 0 (zero) and  
remove tubing



## Safety information and precautions

- Keep the LIFE LINE™ kit and medical oxygen cylinder away from naked flames and sources of heat. Medical oxygen is a non-flammable gas, but it does strongly support combustion
- Ensure medical oxygen cylinders are stored in a safe and secure area where they cannot fall over and cause injury
- Ensure the LIFE LINE™ kit and medical oxygen cylinder are stored in a well-ventilated area
- Never use excessive force when opening or closing the cylinder using the black handwheel
- Never paint the cylinders. All labels and markings must remain clearly visible
- Do not use oil or grease (or any oil-based products which includes hand creams) on or in the vicinity of the medical oxygen cylinder. Combining medical oxygen and oil/grease can cause spontaneous combustion

If you require a replacement medical oxygen cylinder for your LIFE LINE™ kit or have any general enquiries, please contact BOC Customer Service Centre on **1300 363 109**.

## BOC Australia

ABN 95 000 029 729

T: 1300 363 109

W: [www.inhalo.com.au](http://www.inhalo.com.au)

W: [www.boclifeline.com.au](http://www.boclifeline.com.au)

Riverside Corporate Park  
10 Julius Avenue  
North Ryde, NSW 2113  
Australia

BOC is a trading name of BOC Limited, a member of The Linde Group. © BOC Limited 2016. Reproduction without permission is strictly prohibited. Details given in this document are believed to be correct at the time of printing. Whilst proper care has been taken in the preparation, no liability for injury or damage resulting from its improper use can be accepted.

BOC: Living healthcare